

Infinity[®] Kappa Patient Monitor

Vital signs monitor that supports all patients from adults to neonates, in all acuity levels. Intended for fixed monitoring at the bedside, the standard monitor includes a CPU base unit that is compatible with standard medical-grade, flat-panel displays.



FEATURES

- Works as a standalone device or connects to Infinity Network via DirectNet or wireless adapter for wired or wireless networking
- Scales using Infinity pods and software options

Monitoring Capabilities

Neonatal, pediatric and adult applications

TECHNICAL DATA

SUPPORTED PARAMETERS

ECG

Displays up to 12 leads


Available leads I, II, III, aVR, aVF, aVL, V, V+, V1 – V6 [V, aVR aVF, aVL only with 5- and 6-lead sets, V+ only with 6-lead set, V1 to V6 only with 12-lead pod (12-lead not intended for neonates)], TruST[®] 12-lead with reduced lead-set (6-wire): I, II, III, aVL, aVR, aVF, dV1, V2, dV3, dV4, V5 and dV6 (indicated for adults and pediatrics).¹

Measuring range (heart rate) 15 to 300 bpm

Accuracy ± 2 bpm or $\pm 1\%$ (whichever is greater)

Frequency ranges Filter off: 0.05 to 40 Hz display; 0.05 to 125 Hz printer
Monitoring filter: 0.5 to 40 Hz; ESU filter: 0.5 to 16 Hz

¹ Optimum performance of TruST leads is based on a minimum 0.3mV amplitude and QRS duration <180 milliseconds on patients with a body surface area (BSA) of 1.5 – 2.5 m². TruST 12-lead reduced lead-set ECG algorithm provides 12-lead monitoring using a standard 6-wire lead-set and standard lead placement for limb leads, V2 and V5. ARIES software option enhances TruST 12-lead monitoring with the addition of 12-lead ST Analysis.

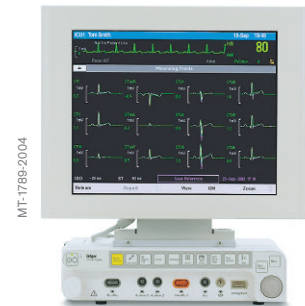

Infinity Kappa
Provides fixed monitoring at the bedside, with a built in power supply.

CONTINUING TECHNICAL DATA**QRS Detection Range**

Amplitude	0.5 to 5 mV
Duration	Adult and pediatric: 70 to 120 msec Neonatal: 40 to 120 msec
Alarms	User-selectable upper and lower limits
Pacer detection (adult/pediatric)	Leads: I, II or III Amplitude: ± 2 to ± 700 mV Width (d.): 0.2 to 2.0 msec
Accessories	3-, 5- or 6-lead electrode set or 12-lead pod

ST (not intended for neonates)

Available leads	With 3-lead ST option: Choice of any 3 available leads With ARIES option: Up to 12 leads
ST complex length	892 msec (-300 to +600 msec from fiducial point)
Sample rate	225 samples/sec
Frequency response	0.05 – 40 Hz

Isoelectric measurement point

Measuring range	Start of ECG complex to fiducial point
Default	QRS onset – 28 msec

ST measurement point

Adjustment range	Fiducial point to end of ECG complex
Point default	QRS offset +80 msec
Update interval	15 sec, 1 normal beat required
Resolution	± 0.1 mm
Trends	Graphical, tabular and graphical mini-trends
INOP Alarm	Yes
Upper and lower ST alarms	± 15 mm, ± 0.1 mm increments
Duration of ST event to trigger alarm	None, 15, 30, 45, 60 seconds

Arrhythmia Detection

Adult and Pediatric	Yes
Neonatal	No, only bradycardia is available as a low heart rate alarm in neonatal mode
ARR Mode	User Selectable; OFF, Basic or Advanced
Basic ARR (standard)	Asystole, ventricular fibrillation, ventricular tachycardia and artifact (ARR label displayed to register arrhythmia occurrence)
Advanced ARR (option)	Ventricular run, accelerated idioventricular rhythm, supra-ventricular tachycardia, couplet, bigeminy, tachycardia, bradycardia, pause and also supports PVC/min parameter output.

Respiration

Sensing leads	I, II (user-selectable)
Measuring method	Impedance pneumography
Auxiliary current	$\leq 10\mu\text{A}$ for any active electrode
Detection threshold	0.15 Ω to 4.0 Ω in manual mode (user adjustment) 0.2 Ω to 1.5 Ω in auto mode (automatic adjustment)
Measuring range	0 to 155 breaths per min
Accuracy	± 1 breath/min or 2% of rate (whichever is greater)
Apnea detection	For neonatal and pediatric patients
Alarms	User-selectable upper and lower respiration rate

Pulse Oximetry (SpO₂)

SpO ₂ algorithm	Masimo® SET® (Signal Extration Technology) Masimo provides the industry gold standard for motion tolerant pulse oximetry. See additional product datasheet for more detailed specifications.
SpO ₂ algorithm	Nellcor™ OxiMax™ ² See additional product datasheet for more detailed specifications.
SpO ₂ algorithm	Dräger's OxiSure® SpO ₂ ³
Dräger's OxiSure® SpO₂	
Connection	MultiMed® pods (SpO ₂ port)
Displayed parameters	Saturation (fraction of oxyhemoglobin to functional hemoglobin) and pulse (rate and waveform)
Measuring method	Transmission spectrophotometry
Measuring range	SpO ₂ : 1 to 100% Pulse: 30 to 250 bpm
Accuracy	SpO ₂ : 0 to 69% not specified SpO ₂ : 70 to 100%: ±2% (±3% for neonates; Masimo® LNOP-Ear: ±3.5%; Nellcor® DS100A: ±3%) Pulse: ±3 bpm or ±3% (whichever is greater)
Alarms	User-selectable upper and lower limits for SpO ₂ and pulse rate Life-threatening desaturation alarm in neonatal mode only
Accessories	Dräger Medical-approved Masimo or Nellcor sensors Dräger Medical reusable SpO ₂ probes (not intended for neonates).

Temperature

Displayed parameters	Absolute and delta temperatures
Measuring range	Absolute: -5° C to 50° C Delta: 0° C to 55° C
Resolution	0.1° C
Accuracy	Absolute: ±0.1° C Delta: ±0.2° C
Alarms	User-selectable upper and lower limits for absolute and delta values
Accessories	Dräger Medical-approved core and skin probes

Noninvasive Blood Pressure (NBP)

Displayed parameters	Systolic, Mean and Diastolic pressures
Measuring method	Oscillometric utilizing step deflation
Modes of operation	Manual (single measurement); Continuous (5 minutes) and Interval
Interval times	1, 2, 2.5, 3, 5, 10, 15, 20, 25, 30, 45, 60, 120 and 240 minutes
Heart rate measuring range	30 to 240 bpm

Pressure measuring range

Adult	Systolic: 30 to 250 mmHg Mean: 20 to 230 mmHg Diastolic: 10 to 210 mmHg
Pediatric	Systolic: 30 to 170 mmHg Mean: 20 to 150 mmHg Diastolic: 10 to 130 mmHg
Neonatal	Systolic: 30 to 130 mmHg Mean: 20 to 110 mmHg Diastolic: 10 to 100 mmHg

CONTINUING TECHNICAL DATA**Cuff pressure**

Default inflation pressure	
Adult	160 mmHg \pm 10 mmHg
Pediatric	120 mmHg \pm 10 mmHg
Neonatal	110 mmHg \pm 10 mmHg
Inflation pressure after a valid measurement	
Adult	(Last Systolic +25 mmHg) \pm 10 mmHg
Pediatric	(Last Systolic +25 mmHg) \pm 10 mmHg
Neonatal	(Last Systolic +30 mmHg) \pm 5 mmHg
Maximum inflation pressure	
Adult	265 mmHg \pm 5 mmHg
Pediatric	180 mmHg \pm 10 mmHg
Neonatal	142 mmHg \pm 10 mmHg
Minimum inflation pressure	
Adult	110 mmHg \pm 10 mmHg
Pediatric	90 mmHg \pm 10 mmHg
Neonatal	70 mmHg \pm 10 mmHg
Connector	Quick-release connector with single airway

Invasive Blood Pressure

Displays up to 8 pressures	
Measuring method	Resistive strain gauge transducer
Display resolution	1 mmHg
Measuring range	-50 to 400 mmHg (after zeroing)
Frequency ranges	DC to 8 Hz, DC to 16 Hz, or DC to 32 Hz (user-selectable)
Zero balance range	\pm 200 mmHg
Transducer specifications	Dräger Medical-approved transducers with a resistance of 200 to 3000 Ω and an equivalent pressure sensitivity of 5 μ V/V/mmHg \pm 10%
Accuracy	\pm 1 mmHg or \pm 3%, exclusive of transducer (whichever is greater)
IBP alarms	User-selectable upper and lower limits for systolic, mean and diastolic pressures
Accessories	Dräger Medical-approved pressure transducers

Cardiac Output

Parameter display	Cardiac output, Blood Temperature, Injectate Temperature
Measuring method	Thermodilution
Connection	Hemo 2, Hemo 4 or HemoMed™ pods

Measuring range

Cardiac output	0.5 to 20 L/min
Blood temperature	25° C to 43° C (77° F to 109° F)
Injectate temperature	-5° C to +30° C (23° F to 86° F)

Accuracy

Cardiac output	\pm 5% (with 0° C injectate)
Injectate temperature	\pm 0.25° C
Degree of protection against electric shock	Type CF
Defibrillation protection	Defibrillation-Proof Applied Part per IEC 60601-1

DISPLAY SPECIFICATIONS

Type	Medical grade independent TFT active matrix LCD (available sizes 15", 17" or 19")
Channels	4 standard, 6, 8 optional
Resolution	800 x 600 pixels
User interface	Easy-to-use menu structure with rotary knob and fixed keys
Sweep Speed	6.25, 12.5, 25 and 50 mm/s \pm 10% (accuracy only guaranteed for a 15" display)

Alarms

Priorities	3; High (Life Threatening), Medium (Serious), Low (Advisory)
Audio alarm tones	User Selectable: Infinity, IEC 1 ² or IEC 2 ²

Connections

MultiMed[®] cables, Masimo SET[®] SmartPod[®], Nellcor OxiMax SmartPod², HemoMed[™] Pod, 3 Pod
Communication ports, NBP Input, analog output, QRS sync output, RS 232, remote keypad, and
Scio[®] Four modules.

Analog Output

Signals	ECG, arterial blood pressure
Delay	\leq 25 msec

Infinity Network

Networking method	DirectNet or Wireless
Wireless encryption	None, WEP, WPA2 ²

See individual product datasheets for detailed information.

Physical Specifications

Cooling	Fan
Size H x W x D	102 x 368 x 368 mm (4 x 14.5 x 14.5 in)
Weight	8.4 kg (19 lb)

Information Management Capabilities

Data storage	24 hours of trended parameter information
Data resolution	30-second sampling
Trend tables	1-, 5-, 15-, 30- or 60-minute display formats
Trend graphs	1-, 2-, 4-, 8-, 12- or 24-hour display formats

Electrical Specifications

Power consumption	\leq 70 Watts (fully loaded)
Patient leakage current	\leq 10 μ A
Protection class	Internal Class 1 power supply (per IEC 60601-1)
Power requirements	100 to 240 V AC, 3 A
Frequency	50 to 60 Hz
Chassis leakage current	300 μ A @ 120 V AC 500 μ A @ 240 V AC

CONTINUING TECHNICAL DATA

BATTERY SPECIFICATIONS

Internal battery	Battery type: lithium-ion Battery capacity: 20 minutes
Charging time	3.5 hours at 25° C
Mode of operation	Continuous with external power supply; for a limited time with battery backup

Battery capacity varies with parameter configuration. The battery capacity specified above is under the following load conditions: MultiMed with SpO₂ sensor, 2 temperature probes, HemoMed pod with 4 IBP transducers and a catheter, NBP taking measurements every 15 minutes, and no continuous tone being generated. Advanced Communication Options are not supported during battery operation.

Battery capacity may diminish after extended use.

Environmental Requirements

Temperature range

Operating	10° C to 45° C (50° F to 113° F)
Storage	-15° C to 50° C (-5° F to 122° F)

Relative humidity

Operating	20% to 90%, non-condensing
Storage	10% to 95% (with packaging)

Atmospheric pressure

Operating	525 to 795 mmHg (70 to 106 kPa)
Storage	375 to 795 mmHg (50 to 106 kPa)

Standards

IEC 60601-1(2nd edition) and applicable particular and collateral standards,

IEC 60601-1-2:2007, Electromagnetic compatibility CISPR 11, Class B

The monitor complies with Medical Devices Directive (MDD) 93/42 EEC and bear the CE mark.

ORDERING INFORMATION

Monitor	MS18853
---------	---------

Note: power cable, MultiMed, and all patient connection and intermediate cables must be ordered separately.

Power Cables

Europe, CEE 7, 2.5 m	4321712
North America, 5-15R, 2.25 m	4321720
Switzerland, SEV 1 01 1, 2.25 m	4321613
Great Britain, BS 1363, 3 m	1851713
Australia, New Zealand, AS3112, 3 m	1851705
China, AS 3112, 3 m	1859714
Denmark, 3 m	1851721

MultiMed Pods and Cables

Multi-parameter cables to monitor

3-, 5- or 6- lead ECG, impedance respiration, SpO₂* and one temperature (two temperatures with Y-cable)

MultiMed Plus, 2.5 m	MS20073
MultiMed Plus OR, 2.5 m	MS20074
Includes integrated ESU filter for operating room environments.	
MultiMed 5, 2.5 m	3368391
MultiMed 6, 2.5 m	5191221
NeoMed, 2.5 m	5590539
3-lead ECG, impedance respiration, two temperatures, SpO ₂ * and FiO ₂ .	



MT-1126-2007

MultiMed Pod

MT-2185-2003

HemoMed Pod

MT-9018-2006

Recorder**CONTINUING ORDERING INFORMATION**

MultiMed or NeoMed Pole/Rail Mount	MP00721
MultiMed 12 Pod ⁵	5589663

For diagnostic 12-lead ECG and SpO₂*

*SpO₂ measurements are not available from the MultiMed pods and cables if you are using an alternate source of SpO₂.

SpO₂ Pods

Masimo SET SpO ₂ SmartPod ⁵	MS16901
Nellcor OxiMax SpO ₂ SmartPod ^{2, 5}	MS23997

Software Options

5 waveform channel option	5597211
6 waveform channel option	5597914
6 - 8 waveform channel option	5597922
Physiological Calculations option ⁵	5201996
Arrhythmia II option (ACE)	4322967
Wireless option**	7498087
3-lead ST Analysis Option (not required with 12-lead option)	5201988
ARIES 12-lead ST analysis option	5597328
ARIES/Physiological Calcs/Arrhythmia package	5443910
OR mode option	MS17653
Advanced Communication Option	7261246

Provides memory for 4 additional user-defined setups, surgical display controller support, up to 4 MIB connections (requires 7487205)

**Wireless LAN PC card (MS25009²), and access point installation is required for wireless monitoring.

Optional Modules and Hardware Accessories

Invasive Blood Pressure Adapters	
2 IBP Y-adapter, 10-pin	5731281
2 IBP Y-adapter, 7-pin	5592147

Hemodynamic Pods

HemoMed Pod ⁵	5588822
Provides management of up to 4 invasive blood pressures and cardiac output.	
Hemo2 [®] Pod ⁵	4319435
Provides management of up to 2 invasive blood pressures, cardiac output and 1 additional temperature.	
Hemo4 [®] Pod ⁵	4315961
Provides management of up to 4 invasive blood pressures, cardiac output and 1 additional temperature.	

PiCCO[®] SmartPod⁵ Kit

PiCCO SmartPod Kit	MS16734
--------------------	---------

PiCCO technology uses quantitative parameters that are determined both intermittently through PULSION's transpulmonary thermodilution technique and continuously through arterial pulse contour analysis.

Provides management of up to 4 invasive blood pressures.

PULSIOCATH arterial thermodilution catheters can be procured from Pulsion directly.

etCO₂, Transcutaneous O₂/CO₂ Gas Monitoring

etCO ₂ pod (Mainstream/Sidestream) ⁵	5740738
etCO ₂ Microstream [®] pod ⁵	7870947
etCO ₂ + Respiratory Mechanics pod ⁵	5740704
topO ₂ /CO ₂ pod ⁵	5592535
Scio Four Modules ⁵	6871810
Scio Four Oxi plus, Scio Four plus, Scio Four Oxi and Scio Four modules	

CONTINUING ORDERING INFORMATION**Neurological Monitoring**

EEG Pod ⁵	5736744
Trident [®] (NMT) SmartPod ⁵	MS15007
BISx™ SmartPod ⁵	MS14796

Printing/Recording Options

R50 recorder ⁵	5952630
R50N network recorder ⁵	5740068
Infinity Network laser printer (115 V)	6556513
Infinity Network laser printer (220 V)	6556539

Other Accessories

Remote keypad	5203042
---------------	---------

Displays

15" flat panel display	MS24381
17" flat panel display	MS24382
19" flat panel display	MS24383
Display mounting kit	MS14583

² Requires VF8 software³ Certain markets do not offer this algorithm⁴ Only available with Dräger's OxiSure algorithm.⁵ Refer to individual module or pod datasheet for additional information.

BISx is a trademark of Aspect Medical Systems, Inc., and is registered in the USA, EU and other countries. The Infinity BISx pod is manufactured by Aspect Medical Systems.

Masimo and SET are registered trademarks of Masimo Corporation

MicroStream is a registered trademark of Oridion

Nellcor and OxiMax are trademarks of Covidien; Covidien with logo and TM marked brands are trademarks of Covidien AG or its affiliate.

PiCCO, Pulsion, and PULSIOCATH are registered trademarks of PULSION Medical Systems AG

HEADQUARTERS

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany

www.draeger.com

REGION EUROPE CENTRAL AND EUROPE NORTH

Dräger Medical GmbH
Moislinger Allee 53–55
23558 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION EUROPE SOUTH

Dräger Médical S.A.S.
Parc de Haute
Technologie d'Antony 2
25, rue Georges Besse
92182 Antony Cedex, France
Tel +33 1 46 11 56 00
Fax +33 1 40 96 97 20
dlmfr-contact@draeger.com

REGION MIDDLE EAST, AFRICA, CENTRAL AND SOUTH AMERICA

Dräger Medical GmbH
Branch Office Dubai
Dubai Healthcare City, P.O. Box 505108
Dubai, United Arab Emirates
Tel + 971 436 24 762
Fax + 971 436 24 761
contactuae@draeger.com

REGION ASIA / PACIFIC

Dräger Medical
South East Asia Pte Ltd
25 International Business Park
#04-27/29 German Centre
Singapore 609916, Singapore
Tel +65 6572 4388
Fax +65 6572 4399
asia.pacific@draeger.com

REGION NORTH AMERICA

Dräger Medical, Inc.
3135 Quarry Road
Telford, PA 18969-1042, USA
Tel +1 215 721 5400
Toll-free +1 800 437 2437
Fax +1 215 723 5935
info.usa@draeger.com

Manufacturer:

Dräger Medical Systems, Inc.
Telford, PA 18969, USA
The quality management system at Dräger Medical Systems, Inc. is certified according to ISO 13485, ISO 9001 and Annex II.3 of Directive 93/42/EEC (Medical devices).